STERILE NITI ENGINE FILES

1. Description

Sterile NiTi engine Files are type 2 rotary dental instruments conforming to EN ISO 3630-1.

The Sterile NiTi Engine files are composed of operative part, shank and stopper, which needs to be used in combination with handpiece of type 3 in compliance with ISO 1797 standard.

The chuck of the handpiece shall be fitted to Type I which is met ISO 1797 requirement.

These products are sterilized by gamma irradiation and supplied in a sterile blister.

Single use products.

2. Intended use

Sterile NiTi Engine files are endodontic instruments intended to remove dentine to prepare and shape root canal for the treatment of various pulpitis and root canal diseases with dental necrosis.

3. Users

The product is only used by dentists-surgeons who have received formal training.

4. Patient population

Sterile NiTi Engine are intended to be used on adults and children with a tooth affected by pulpitis or pulp necrosis.

5. Indications

This device is indicated for the root canal preparation during non-surgical endodontic treatment and retreatment.

6. Contraindications

The product contains Nickel and Chromium and should not be used on patients with known allergic sensitivity to these metals.

7. Complications

In the complex anatomy, intraoperative risks (device fracture, steps, lateral wall perforation, open root tip, wrong passage, perforation...) and the risk of causing infection may occur.

In the present technical state, no adverse reaction has been reported so far.

8. Characteristics

See tables in annex.

9. Warnings

For your own safety, please wear personal protective equipment (gloves, glasses, mask).

Check whether the product is within the prescribed validity period before use.

The engine root canal file is used together with motor, with an allowable speed range of 250-1000rpm. Use the indicated rotation method at the recommended speed of the label or refer to §8 "characteristics".

In anatomical complexity, the maximum torque can differ from the manufacturer's recommended value.

These products are sterilized by gamma irradiation, which is one single use only, second use is prohibited. If reused, it may increase the risk of patient infection and device break.

The type of sterile NiTi file to be used or selected will depend on the practitioner, in function of the anatomy of the root canal and the complexity of the case.

Follow the recommended usage steps for use in §13.

Keep the UDI information on the label.

Do not use the instruments when storage conditions are suspected.

If you have any questions about the product identity, do not use it.

Check the instrument state before the use between each canal. If the instrument is damaged or shows signs of wear, do not use the instrument.

Serious accidents related with this product shall be reported to the manufacturer and national authorities.

These products are sharp, take protection measurements, to avoid injury risks while take up and use it.

10. Cautions

These products are only used by dentists-surgeons.

This product is limited to dental treatment purposes and is prohibited for operations other than dental treatment.

Follow good dentist practice, especially using the rubber dam system during the root canal treatment.

These products are under sterile packaging and should check for damage before use. If abnormal, do not use it.

11. Technical performances

- The torsional strength (torque): 0.34 mN·m,
- Deflection angle: 120°
- Bending strength (bending torque): 50.96 mN·m.

12. Indirect Clinical Benefits

The following clinical benefits are claimed for the medical device:

- The use of root canal files allows the dentist to save a tooth from extraction by removing the dentin and shaping of the root canal.
- The treatment use root canal files helps alleviate toothaches and other discomfort associated with tooth decay or infection

13. Use steps

1. Verification of chosen file specifications.
2. Connection of the shank of engine files to the Handpiece of contra-angle of endo motor.
3. Ensure that the rubber pad is correctly fixe and not moveable.
4. Insert the file into patient's root canal.
5. Once canal preparation is complete, switch off the handpiece.
6. Remove the file of patient's root canal.
7. Disconnect the file from the handpiece.
8. Dispose of the file in a container provided for this purpose.

14. Disposal of the device

After use, follow good practice for dental sterile NiTi Engine files and must be placed in a solid container that should be used for collecting cutting or bonding instruments.

15. Product storage conditions

To avoid damaging the packaging materials, handle with care during transport and storage.

Avoid storage in high humidity and direct sunlight, and the products should be stored at room temperature.

16. Symbol identification

Manufacturer

Importer

Swiss representative

Medical device

Date of manufacture

Sterilised by irradiation

Expiry date

Do not use if the packaging is damaged

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