1. Specifications

Model: SO25, SO40, SO50
Length: 21mm, 25mm, 31mm
Package: Plastic box-6pcs/box

2. Raw materials

NiTi alloy

3. Function

This is the dental instrument with columnar or tapering working part and it possesses circular form in cross cut section and flat end. REVERSO SILVER files are used aiming at shaping and cleaning the root canal by mechanically powered REVERSO SILVER system.

4. Recommend Speed and Torque

Model	Length (mm)	Taper	Speed (RPM)	Torque
SO25	21, 25, 31	8%	250-350	3.5
SO40	21, 25, 31	6%	250-350	3.5
SO50	21, 25, 31	5%	250-350	3.5

5. Motion

Reciprocating movement

6. Precautions for use

Safety equipment (gloves, glasses, rubber dam) is required for patients and self-protection.

These instruments can only be used after sterilization, please using these files under medical environment.

Please use REVERSO SILVER system in an appropriate case and in a correct way. Remove debris out frequently from screwing-in problems

Normally shaping the canal with EDTA for entering the canal smoothly and irrigate sufficiently after removing debris created by shaping the root canal.

If you come across a situation that it's difficult to enter the deeper part of the canal, please stop and check both morphology of the canal and whether there are signs of deformation of files or not, then confirm the orifice is expanded wide enough for subsequent use of shaping files.

7. Selecting the correct REVERSO SILVER

- Normally, 25# for most of case
- #40 for medium canal
- #50 for bigger canal

8. Sequence

- To achieve a straight-line access to the root canal entrance.
- Irrigating the root canal
- Select the correct REVERSO SILVER instruments
- Check the Endo motor has been selected
- Introduce the REVERSO SILVER into the root canal.
- Move the instrument in a slow in-and-out pecking motion. The amplitude of the in-and-out movements should not exceed 3 mm. Only light pressure should be applied. 1 in-and-out movement=1 peck. The instrument will advance easily in the canal.
- After 3 pecks, remove the instrument from the canal. Clean the debris.
- Irrigate the canal.
- Make sure the canal is free to approx 3mm beyond the prepared canal section with an ISO size 10 file.
- In this way, continue with REVERSO SILVER instrument until approx 2/3 of the working length has been reached. When using an 25#, determine the working length by using an ISO size 10# files. When using an 40#, 50#, the working length should be re-checked with an apex locator.
- Continue with REVERSO SILVER instrument until working length has been reached.
- As soon as full working length has been reached, withdraw the instrument from the root canal.

9. Technique tips

- REVERSO SILVER instruments can be used in a lateral brushing motion to enable preparation of irregular shaped canals of to enlarge the root canal entrance. The lateral brushing motion can also help to achieve an easier advancement of the instrument.
- Clean REVERSO SILVER instrument flute after 3 pecks.
- Use an ISO size 10# file to check the situation of the canal, confirm the canal is not blocked after 3 pecks with REVERSO SILVER.
- Irrigate the root canal according to the appropriate irrigation protocol.
- Never apply pressure if resistance is encountered, instead clean the files and check the canal with an ISO file size 10#.
- Remove the instrument from the canal as soon as working length has been reached. Working with engine files too long in one spot will cause canal transportation.
- Pull REVERSO SILVER out of the canal after 3 pecks or when resistance is encountered.

10. Cleaning and sterilization

Method of sterilization:

Put this product in a sterilization pack (or foil) and place it on a sterilization tray for autoclave sterilization with reference to the following terms:

1. Sterilize with steam at 134°C (273°F) / 2,1 bars / 18 min. for endodontic instruments and filling.
2. Do not use high pressure steam sterilizer which heats more than 200 degrees Celsius including drying process.
3. When reusing the instrument, wash away foreign substances completely and sterilize.
4. Regarding use of medical cleaning agent, follow the instruction manual by its manufacturer strictly.
5. Dispose the product if damaged or decreased in performance.

11. Warnings

- a. Only skilled dentists are allowed to use.
- b. Be sure to sterilize this product for each use.
- c. Do not use this product except for the dental service and treatment. Use it in accordance with the intended use.
- d. Please confirm that patient has no allergy to Nickel Titanium alloy and other medical reaction before the treatment.
- e. These instruments are intended for use only in dental root canal treatment, users must follow a correct and reasonable way under medical environments.
- f. Please use a rubber dam system to ensure patients' safety when deliver endodontic treatment.
- g. Do not use this product for a patient who indicates sensitization and allergic reaction.

12. Storages and duration of use

- a. Avoid storing at high temperature, humidity and direct sunlight. Keep liquids away. Store it at room temperature.
- b. Do not damage or make a pinhole to packaging materials.
- c. This product is subject to be improved without previous notice. Enforce first-in first-out method for stock management.
- d. Do not store the instrument under a germicidal lamp to avoid deterioration.

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DESIGNED IN FRANCE

**Shenzhen SuperLine
Technology Co. Ltd.**
Room 1206, Building 4 Chongwen Garden,
Taoyuan Street Nanshan District
Shenzhen 518055 Guangdong China

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Wellkang Ltd. Suite B,