## EASY-FILE FLEX

## 1. Specifications

Model: S020, R25, M35, L45

Length: 21mm, 25mm, 31mm

Package:

- Sterile blister assorted kit containing - 4 files (1 of each model)
- Sterile blister - 6 files of the same reference

## 2. Raw Materials

NiTi alloy with heat treatment

## 3. Function

This is the dental instrument with columnar or tapering working part and it possesses circular form in cross cut section and flat end. EASY-FILE FLEX files are used aiming at shaping and cleaning the root canal by mechanically powered EASY-FILE FLEX system.

## 4. Recommend Speed and Torque

| Model | Length (mm) | ISO | Taper | Speed (RPM) | Torque (N.CM) |
|-------|-------------|-----|-------|-------------|---------------|
| S20   | 21, 25, 31  | 020 | 7%    | 250-350     | 3.5           |
| R25   | 21, 25, 31  | 025 | 7%    | 250-350     | 3.5           |
| M35   | 21, 25, 31  | 035 | 6%    | 250-350     | 3.5           |
| L45   | 21, 25, 31  | 045 | 5%    | 250-350     | 3.5           |

## 5. Motion

Alternative movement (anti-clockwise and clockwise)

## 6. Precautions for use

- Safety equipment (gloves, glasses, rubber dam) is required for patients and self-protection.
- These instruments can only be used after sterilization, please using these files under medical environment.
- EASY-FILE FLEX system are all heat-treated instruments and showing a greater flexibility performance, it's a normal phenomenon that it might appear a little bent but it won't bring any disturbance for use.
- Reciprocating motion is mandatory to match the EASY-FILE FLEX system.
- Please use EASY-FILE FLEX system in an appropriate case and in a correct way. Remove debris out frequently from screwing-in problems
- Normally shaping the canal with EDTA for entering the canal smoothly and irrigate sufficiently after removing debris created by shaping the root canal.
- If you come across a situation that it's difficult to enter the deeper part of the canal, please stop and check both morphology of the canal and whether there are signs of deformation of files or not, then confirm the orifice is expanded wide enough for subsequent use of shaping files.
- EASY-FILE FLEX files are supplied sterile and reuse may increase the risk of cross-contamination or breakage.

## 7. Selecting the correct EASY-FILE FLEX

- Easy-flex file are: 20,25,35,45. There is no size, 40, 50#
- 25# for most of case, 35#,45# for bigger canal, 20# is for small canal
- If ssk 10# goes into canal and feel tigh, then use 20/07 as first shaping file
- If SSK 10# goes into canal and feel easy, then use 25# as first shaping file
- If ssk, 20# goes into canal and feel easy, then use 35# or 45# as first shaping file

## 8. Sequence

### Preoperative radiographic evaluation

Observe the configuration of canal, then confirm the quantity, width, length, curvature, and choose a most suitable file system.

### Access

Prepare a straight access to canal orifice.

### Scouting

Explore the canal up to size #10 hand use file with EDTA.

### Smooth path done by EASY-FILE FLEX GLIDER

Create a smooth path and confirming the whole working length can be totally reached after the orifice is well expanded in order to be well prepared for subsequent use of shaping files, as a excellent connection between hand use files and rotary files.

### Shaping with EASY-FILE FLEX

On the basis of created glide path, shape the canal with EASY-FILE FLEX in the condition of EDTA. Normally choose PRIMARY file (25/07) as the initial file to start, changing the most optimal file subject to morphology of the canal.

## 9. Cleaning and sterilization

### Sterilisation method for re-use:

**Put this product in a sterilization pack (or foil) and place it on a sterilization tray for autoclave sterilization with reference to the following terms:**

1. Sterilize with steam at 134°C (273°F) / 2,1 bars / 18 min. for endodontic instruments and filling.
2. Do not use high pressure steam sterilizer which heats more than 200 degrees Celsius including drying process.
3. When reusing the instrument, wash away foreign substances completely and sterilize.
4. Regarding use of cleaning agent, follow the instruction manual by its manufacturer strictly.
5. Dispose the product if damaged or decreased in performance.

## 10. Warnings

- a. Only skilled dentists are allowed to use.
- b. Be sure to sterilize this product for each use.
- c. Do not use this product except for the dental service and treatment. Use it in accordance with the intended use.
- d. Please confirm that patient has no allergy to Nickel Titanium alloy and other medical reaction before the treatment.
- e. These instruments are intended for use only in dental root canal treatment, users must follow a correct and reasonable way under medical environments.
- f. Please use a rubber dam system to ensure patients' safety when deliver endodontic treatment.
- g. Do not use this product for a patient who indicates sensitization and allergic reaction.

## 11. Storages and duration of use

- a. Avoid storing at high temperature, humidity and direct sunlight. Keep liquids away. Store it at room temperature.
- b. Do not damage or make a pinhole to packaging materials.
- c. This product is subject to be improved without previous notice. Enforce first-in first-out method for stock management.
- d. Do not store the instrument under a germicidal lamp to avoid deterioration.

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Wellkang Ltd.
Technology Co. Ltd.**