1. Specifications

Length: 21mm, 25mm, 31mm
 Package: Sterile blister - 6 files

2. Raw materials

NiTi alloy

3. Function

This instrument will prepare the passage of “root canal preparation or shaping” files (GO-TAPER, GO-TWO, REVERSO or EASY-FILE FLEX type).

Fits to all endodontic motors and contra-angles with rotary movement.

4. Recommend Speed and Torque

Model	Length (mm)	ISO	Taper	Speed (RPM)	Torque (N.CM)
GO-GLIDER	21, 25, 31	016	2%	300	2.5

5. Motion

Continuous rotation

6. Precautions for use

- Safety equipment (gloves, glasses, rubber dam) is required for patients and self-protection.
- These instruments can only be used after sterilization, please using these files under medical environment.
- Please use GO-GLIDER system in an appropriate case and in a correct way. Remove debris out frequently from screwing-in problems
- Normally shaping the canal with EDTA for entering the canal smoothly and irrigate sufficiently after removing debris created by shaping the root canal.
- If you come across a situation that it's difficult to enter the deeper part of the canal, please stop and check both morphology of the canal and whether there are signs of deformation of files or not, then confirm the orifice is expanded wide enough for subsequent use of shaping files.
- GO-GLIDER files are supplied sterile and reuse may increase the risk of cross-contamination or breakage.

7. Operating protocol

After opening the tooth, fill the entire pulp chamber with sodium hypochlorite solution. Penetrate the canals with a #10 file until resistance is felt. Measure this length and prepare the canal to this length using the GO-GLIDER. Rinse thoroughly with sodium hypochlorite. Take the #10 file and move it in the canal to the working length. Measure the length and prepare the canal to this length using the GO-GLIDER. Rinse and finish the root canal preparation root canal preparation or shaping files to the working length. (EASY-FILE FLEX or REVERSO BLUE or REVERSO SILVER or GO-TAPER FLEX or GOTAPER UNIVERSAL).

8. Technique tips

- Using GO-GLIDER with a torque control, electric endo motor
- Before using GO-GLIDER, glide path shall be created
- After each rotary files, irrigate, rinsing to clean the canals
- Frequently inspect and clean the flutes of rotary files

9. Cleaning and sterilization

Sterilisation method for re-use:

Put this product in a sterilization pack (or foil) and place it on a sterilization tray for autoclave sterilization with reference to the following terms:

1. Sterilize with steam at 134°C (273°F) / 2,1 bars / 18 min for endodontic instruments and filling.
2. Do not use high pressure steam sterilizer which heats more than 200 degrees Celsius including drying process.
3. When reusing the instrument, wash away foreign substances completely and sterilize.
4. Regarding use of cleaning agent, follow the instruction manual by its manufacturer strictly.
5. Dispose the product if damaged or decreased in performance.

10. Warnings

- a. Only skilled dentists are allowed to use.
- b. Be sure to sterilize this product for each use.
- c. Do not use this product except for the dental service and treatment. Use it in accordance with the intended use.
- d. Please confirm that patient has no allergy to Nickle Titanium alloy and other medical reaction before the treatment.
- e. These instruments are intended for use only in dental root canal treatment, users must follow a correct and reasonable way under medical environments.
- f. Please use a rubber dam system to ensure patients' safety when deliver endodontic treatment.
- g. Do not use this product for a patient who indicates sensitization and allergic reaction.

11. Storages and duration of use

- Avoid storing at high temperature, humidity and direct sunlight. Keep liquids away. Store it at room temperature.
- Do not damage or make a pinhole to packaging materials.
- This product is subject to be improved without previous notice. Enforce first-in first-out method for stock management.
- Do not store the instrument under a germicidal lamp to avoid deterioration.

6 pcs/box

Continuous rotation

Sterilized using irradiation

Nickel titanium product

For endo motor use

Stoppers are silicon stoppers

Speed

Sterilize at 134° C

Do not use if package is damaged

Manufacturer

Importer

Authorized representative in the European Community

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Wellkang Ltd.
Shenzhen SuperLine Technology Co. Ltd.