# GO-TAPER RETREATMENT

## 1. Specifications

Model: C1,C2,C3

Length: 16mm, 18mm, 22mm

Package: Sterile blister - 6 files

## 2. Raw materials

NiTi alloy

## 3. Function

This is the dental instrument with columnar or tapering working part and it possesses circular form in cross cut section and flat end. GO-TAPER RETREATMENT files are used aiming at shaping and cleaning the root canal by mechanically powered GO-TAPER RETREATMENT system.

## 4. Recommend Speed and Torque

Model	Length (mm)	ISO	Taper	Speed(RPM) for Gutta percha	Speed(RPM) for Obturator based on Zinc oxide eugenol	Torque (N.CM)
C1	16	030	9%	500	250-350	2-5.2
C2	18	025	8%	500	250-350	2-5.2
C3	22	020	7%	500	250-350	2-5.2

## 5. Motion

Continuous rotation

## 6. Precautions for use

- Go-Taper Retreatment files are supplied sterile and reuse may increase the risk of cross-contamination or breakage.
- Repeated disinfection and sterilisation may result in an increased risk of file separation and fracture. fracture of the files.
- This instrument should not be immersed in sodium hypochlorite solution.
- Cleaning the instrument: follow the instructions strictly.

## 7. Selecting the correct GO-TAPER RETREATMENT

- C1 for coronal part
- C2 for middle part
- C3 for apical part

## 8. Sequence

- Build an appropriate hold using a small size hand use stainless steel files by to ultrasonic instruments or heat carrier.
- Press the file C1 into the filling materials with gentle force. Use file C1 for coronal filling materials removal
- Take C2 for middle filling materials removal Get.
- C3 for apical filling materials removal
- Use a hand use file with a solvent to remove filling materials from apical part when encountering intricate anatomy.
- Check blade situation without debris, filling materials, and keep files are clean.
- Continue and finish till the blade without filling materials.

## 9. Technique tips

- X-ray is necessary before retreatment;
- Analysis the root canal carefully;
- Obtain straight line access to the canal;
- Heat will be helpful for filling materials removal;
- Re-treatment file is sterilizable.

## 10. Cleaning and sterilization

### Sterilisation method for re-use:

Put this product in a sterilization pack (or foil) and place it on a sterilization tray for autoclave sterilization with reference to the following terms:

1. Sterilize with steam at 134°C (273°F) / 2,1 bars / 18 min for endodontic instruments and filling.
2. Do not use high pressure steam sterilizer which heats more than 200 degrees Celsius including drying process.
3. When reusing the instrument, wash away foreign substances completely and sterilize.
4. Regarding use of cleaning agent, follow the instruction manual by its manufacturer strictly.
5. Dispose the product if damaged or decreased in performance.

## 11. Warnings

- a. Only skilled dentists are allowed to use.
- b. Be sure to sterilize this product for each use.
- c. Do not use this product except for the dental service and treatment. Use it in accordance with the intended use.
- d. Please confirm that patient has no allergy to Nickel Titanium alloy and other medical reaction before the treatment.
- e. These instruments are intended for use only in dental root canal treatment, users must follow a correct and reasonable way under medical environments.
- f. Please use a rubber dam system to ensure patients' safety when deliver endodontic treatment.
- g. Do not use this product for a patient who indicates sensitization and allergic reaction.

## 12. Storages and duration of use

- a. Avoid storing at high temperature, humidity and direct sunlight. Keep liquids away. Store it at room temperature.
- b. Do not damage or make a pinhole to packaging materials.
- c. This product is subject to be improved without previous notice. Enforce first-in first-out method for stock management.
- d. Do not store the instrument under a germicidal lamp to avoid deterioration.

Assorted package

Do not use if package is damaged

6 pcs/box

Nickel titanium product

Sterilized using irradiation

Stoppers are silicon stoppers

For endo motor use

Sterilize at 134° C

250 -  
350 rpm

Speed

Continuous rotation

Manufacturer

Authorized representative in the European Community

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**Wellkang Ltd.**
**Shenzhen SuperLine Technology Co. Ltd.**