1. Specifications

Model: X1, X2, X3, X4, X5
 Length: 21mm, 25mm, 31mm
 Package: Sterile blister - 6 files

2. Material

NiTi alloy

3. Function

The GO-TAPER X file are used in endodontic treatment for shaping and cleaning the root canal system.

4. Recommend Speed and Torque

Model	Length (mm)	ISO	Taper	Speed (RPM)	Torque (N.CM)
X1	21, 25, 31	017	4%	250 - 300	2
X2	21, 25, 31	025	6%	250 - 300	2
X3	21, 25, 31	030	7%	250 - 300	2
X4	21, 25, 31	040	6%	250 - 300	2
X5	21, 25, 31	050	6%	250 - 300	2

5. Motion

Continuous rotation

6. Instructions for GO-TAPER X file

- X-ray film to estimate the working length of canals
- Create straight line access to canal orifice (access bur or gates drills, etc.)
- Always irrigate and if necessary, expand the glide path using small sized hand files or dedicated mechanical glide path files.
- In the presence of NaOCl, brush and follow, along the glide path, with the X file X1 (017/4%), in one or more passes, alternatively with small sized hand files if necessary, until the working length is reached.
- Use X file X2 (025/6%), exactly as described for X file, until the working length is passively reached.
- Inspect the apical flutes of the GO-TAPER X file X2, if they are loaded with dentin, then the shape is finished, the correspondingly sized gutta percha master cone or size verifier may be fitted, and the canal is ready for disinfection.
- Alternatively, gauge the foramen with a size 025 hand file and if this file is snug at length, the canal is shaped and ready for disinfection.
- If the size 025 hand file is loose at length, then continue shaping with the X file X3 (030/7%) and when necessary, the X file X4, X5, gauging after each instrument with the 030, 040 or 050 hand files, respectively.

7. Warnings

- This product contains Nickel and should not be used for individuals with known allergic sensitivity to this metal.
- In order to prevent infectious agent, transfer it is highly recommended to use a rubber dam system during the endodontic procedure.

8. Precautions:

- Exercise caution in the apical area and around significant curvatures.
- These instruments should not be immersed in a sodium hypochlorite solution.
- NaOCl, EDTA are recommended.
- Establish a glide path using small size hand file or glide path file.
- Use in a constant rotation at speed 300 rpm, 2-5.2 Ncm.
- Clean flutes frequently and check for signs of distortion or wear.
- The GO-TAPER X file are recommended to be used in a clockwise motion, brushing the canal.
- GO-TAPER X files are supplied sterile and reuse may increase the risk of cross-contamination or breakage.

9. Cleaning and sterilization

Sterilisation method for re-use:

Put this product in a sterilization pack (or foil) and place it on a sterilization tray for autoclave sterilization with reference to the following terms:

1. Sterilize with steam at 134°C (273°F) / 2,1 bars / 18 min for endodontic instruments and filling.
2. Do not use high pressure steam sterilizer which heats more than 200 degrees Celsius including drying process.

3. When reusing the instrument, wash away foreign substances completely and sterilize.
4. Regarding use of cleaning agent, follow the instruction manual by its manufacturer strictly.
5. Dispose the product if damaged or decreased in performance.

10. Warnings

- a. Only skilled dentists are allowed to use.
- b. Be sure to sterilize this product for each use.
- c. Do not use this product except for the dental service and treatment. Use it in accordance with the intended use.
- d. Please confirm that patient has no allergy to Nickel Titanium alloy and other medical reaction before the treatment.
- e. These instruments are intended for use only in dental root canal treatment, users must follow a correct and reasonable way under medical environments.
- f. Please use a rubber dam system to ensure patients' safety when deliver endodontic treatment.
- g. Do not use this product for a patient who indicates sensitization and allergic reaction.

11. Storages and duration of use

- a. Avoid storing at high temperature, humidity and direct sunlight. Keep liquids away. Store it at room temperature.
- b. Do not damage or make a pinhole to packaging materials.
- c. This product is subject to be improved without previous notice. Enforce first-in first-out method for stock management.
- d. Do not store the instrument under a germicidal lamp to avoid deterioration.

Assorted package

Do not use if package is damaged

6 pcs/box

Nickel titanium product

Sterilized using irradiation

Stoppers are silicon stoppers

For endo motor use

Sterilize at 134° C

Speed

Continuous rotation

Manufacturer

Authorized representative in the European Community

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Wellkang Ltd.
Shenzhen SuperLine Technology Co. Ltd.