## REVERSO BLUE

## 1. Specifications

Model : SO25, SO40, SO50  
 Length : 21mm, 25mm, 31 mm  
 Package : Sterile blister - 6 files

## 2. Raw materials

NiTi alloy with heat treatment

## 3. Function

This is the dental instrument with columnar or tapering working part and it possesses circular form in cross cut section and flat end. REVERSO BLUE files are used aiming at shaping and cleaning the root canal by mechanically powered REVERSO BLUE system.

## 4. Recommend Speed and Torque

Model	Length (mm)	ISO	Taper	Speed (RPM)	Torque (N.CM)
SO25	21, 25, 31	025	8%	250-350	3.5
SO40	21, 25, 31	040	6%	250-350	3.5
SO50	21, 25, 31	050	5%	250-350	3.5

## 5. Motion

Alternative movement (anti-clockwise and clockwise)

## 6. Precautions for use

- Safety equipment (gloves, glasses, rubber dam) is required for patients and self-protection.
- These instruments can only be used after sterilization, please using these files under medical environment.
- REVERSO BLUE system are all heat-treated instruments and showing a greater flexibility performance, it's a normal phenomenon that it might appear a little bent but it won't bring any disturbance for use.
- Reciprocating motion is mandatory to match the REVERSO BLUE system.
- Please use REVERSO BLUE system in an appropriate case and in a correct way. Remove debris out frequently from screwing-in problems
- Normally shaping the canal with EDTA for entering the canal smoothly and irrigate sufficiently after removing debris created by shaping the root canal.
- If you come across a situation that it's difficult to enter the deeper part of the canal, please stop and check both morphology of the canal and whether there are signs of deformation of files or not, then confirm the orifice is expanded wide enough for subsequent use of shaping files.
- REVERSO BLUE files are supplied sterile and reuse may increase the risk of cross-contamination or breakage.

## 7. Selecting the correct REVERSO BLUE

- If the canal is partially or completely invisible on the radiograph: the canal is considered narrow, use size 25#
- If the canal is completely visible on the radiograph : take an ISO 30# hand file, insert it passively to working length. If it reached working length, the canal is considered large, use 50#.
- If an ISO 30 hand files does not goes passively to working length, try an ISO 20# hand file. If this goes passively to working length, the canal is considered medium, use 40#.

## 8. Sequence

- To achieve a straight line access to the root canal entrance.
- Irrigating the root canal.
- Select the correct REVERSO BLUE instruments
- Check the Endo motor has been selected
- Introduce the REVERSO BLUE into the root canal.
- Move the instrument in a slow in-and-out pecking motion. The amplitude of the in-and-out movements should not exceed 3 mm. Only light pressure should be applied. 1 in-and-out movement=1 peck. The instrument will advance easily in the canal.
- After 3 pecks, remove the instrument from the canal. Clean the debris.
- Irrigate the canal.
- Make sure the canal is free to approx 3mm beyond the prepared canal section with an ISO size 10 file.
- Continue with the REVERSO BLUE instrument until working length has been reached.

- As soon as full working length has been reached, withdraw the instrument from the root canal.

## 9. Technique tips

- Pull the instrument out of the canal after 3 pecks or when resistance is encountered.
- Never apply pressure if the instrument does not advance into the canal.
- Check blade situation without debris, filling materials, and keep files are clean.
- Remove the file from the canal as soon as working length has been reached.
- REVERSO BLUE can be used in lateral brushing motion to enable preparation of irregularly shaped canals.
- It is not recommended to use REVERSO BLUE nor continuous rotary system if the root canal has an abrupt curvature in the apical section. It is recommended to use hand file to explore.

## 10. Cleaning and sterilization

**Sterilisation method for re-use :**

**Put this product in a sterilization pack (or foil) and place it on a sterilization tray for autoclave sterilization with reference to the following terms:**

1. Sterilize with steam at 134°C (273°F) / 2,1 bars / 18 min.for endodontic instruments and filling.
2. Do not use high pressure steam sterilizer which heats more than 200 degrees Celsius including drying process.
3. When reusing the instrument, wash away foreign substances completely and sterilize.
- 4.Regarding use of cleaning agent, follow the instruction manual by its manufacturer strictly.
5. Dispose the product if damaged or decreased in performance.

## 11. Warnings

- a. Only skilled dentists are allowed to use.
- b. Be sure to sterilize this product for each use.
- c. Do not use this product except for the dental service and treatment. Use it in accordance with the intended use
- d. Please confirm that patient has no allergy to Nickel Titanium alloy and other medical reaction before the treatment.
- e. These instruments are intended for use only in dental root canal treatment, users must follow a correct and reasonable way under medical environments.
- f. Please use a rubber dam system to ensure patients' safety when deliver endodontic treatment.
- g. Do not use this product for a patient who indicates sensitization and allergic reaction.

## 12. Storages and duration of use

- a. Avoid storing at high temperature, humidity and direct sunlight. Keep liquids away. Store it at room temperature.
- b. Do not damage or make a pinhole to packaging materials.
- c. This product is subject to be improved without previous notice. Enforce first-in first-out method for stock management.
- d. Do not store the instrument under a germicidal lamp to avoid deterioration.

Alternative movement

Do not use if package is damaged

6 pcs/box

Nickel titanium product

Sterilized using irradiation

Stoppers are silicon stoppers

For endo motor use

Sterilize at 134° C

Assorted package

Manufacturer

Authorized representative in the European Community

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**Wellkang Ltd.**
**Shenzhen SuperLine Technology Co. Ltd.**