access
SODIUM
HYPOCHLORITE

- EN For canal irrigation
- FR Pour l'irrigation canalaire
- ES Para la irrigación de canales
- PT Para a irrigação dos canais
- IT Per l'irrigazione dei canali
- DE Zur Bewässerung der Wurzelkanäle
- NL Voor wortelkanaalbehandelingen

5,25%

DO NOT INJECT
DO NOT INGEST
NE PAS INJECTER
NE PAS INGÉRER

400 ml

LOT

CE
0 5 3 7

Magnum Dental OÜ
Aardla 13
50112 Tartu
ESTONIA

CADENCE
2 bis, chemin du Loup
93290 Tremblay-en-France
FRANCE

REF 3162100

3 665318 002732

Version 1 – 11/2023

EN

Description:

Sodium Hypochlorite is an aqueous sodium hypochlorite solution. At low concentrations dissolve mainly necrotic tissue, at higher concentrations also dissolves vital tissue.

Procedure:

It is used for the irrigation and debridement of root canals during endodontic treatment. Remove smear layer with appropriate irrigant such as EDTA solution or Citric Acid 12.5%. Only for professional use.

Instructions for use:

1. Remove degassing seal before use.
2. Fill syringe with Sodium Hypochlorite (10 ml recommended).
3. Attach irrigation tip securely to syringe.
4. Irrigate as desired.
5. Remove and dispose of syringe and irrigating tip.

Precautions:

1. Read MSDS for material handling and safety information.
2. Irrigation should not be irritant for the apical area.
3. Avoid spilling on surrounding tissues.
4. Avoid spilling on colored fabrics and surfaces as Sodium Hypochlorite is moderately alkaline and has a whitening effect.

Warnings:

1. Caution, caustic!
2. Keep out of the reach of children.
3. Avoid contact with skin and eyes.
4. Do not swallow or inject.
5. Avoid mixing Sodium Hypochlorite with chlorhexidine.

Storage:

Store in a dry area away from heat and direct sunlight.

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