STERILE K-FILES / H-FILES / REAMERS

(Stainless Steel)

Before using the **STERILE K-FILES / H-FILES / REAMERS**, please see the IFU as below.

1. Warnings

- Only skilled dentists are allowed to use.
 - Do not use this product except for the dental service and treatment.
- Use it in accordance with the intended use.

2. Contraindications and prohibitions

Do not use this product for a patient who indicates sensitization and allergic reaction.

3. Structures and principals

- This is the dental file with handle used for enlarging root canal. It is used by the movements of vertical reciprocation or lifting. It is sterile instrument.
- Main materials:
 - Working Cutting part: Stainless steel (including nickel and chromium)
 - Handle: Polybutylene terephthalate (PBT)

K-FILE

H-FILE

REAMERS

4. Purpose of use and effect-efficacy

- **K-FILE / H-FILE:** This is the hand dental instrument for enlarging root canal and making smooth and flat root canal wall through grinding and polishing. It is used by the movements of lifting and vertical reciprocation.
- **REAMERS:** This is the hand dental rotary instrument for debriding by side cut and enlarging root canal.

5. Specifications

Under the provisions of ISO 3630 performance test, product durability for twisting and bending must exceed specified value of our standard.

6. Method of use

According to its intended use, choose the most appropriate **K-FILE / H-FILE / REAMERS** and use it for enlarging and debriding root canal.

7. Precautions for use

1. To prevent infection, if you reuse the product sterilize the product by autoclave and make sure sterilization is completed before using.
2. Choose the most appropriate type for each case and follow the general method.
3. Before using, make sure the instruments outside of oral cavity that there are no deformations, scratches and cracks.
4. If the head of product is thin, long or large, there are possibilities for breaking or twisting. Because of this, be sure to avoid using unreasonable angle and excessive pressure.
5. Wear safety glasses to protect your eyes from damage. Also, wear a dust protective mask to prevent inhaling dust.
6. Wear rubber dam to avoid accidental ingestion and falling.
7. Do not use this instrument for any purposes except for listed applications above.
8. Only for use by dentists.
9. This product should be treated as medical waste when disposed.
10. Dispose the product if damaged or contaminated.
11. After using, wash it with medical cleaning agent and brush, then wash away foreign substances like adherent body fluid and body tissues completely.
12. Set the product to dry and when cleaned by ultrasonic cleaner to avoid working part deterioration.
13. Use this product with great care to avoid puncturing fingers because of its possession of sharp-edged part.
14. Avoid using unreasonable angle and excessive pressure.

15. This product has possibility that be corrosive if sunk into NaOCl, EDTA and etc. for a long time.

8. Storages and duration of use

1. Avoid storing at high temperature, humidity and direct sunlight. Keep liquids away. Store it at room temperature.
2. Do not damage or make a pinhole to packaging materials.
3. This product is subject to be improved without previous notice. Enforce first-in first-out method for stock management.
4. Do not store the instrument under a germicidal lamp to avoid deterioration.

9. Method of sterilization

The product you bought is already sterile.

If you need to re-sterilize the product, put it in a sterilization pouch and place it on a sterilization tray for autoclave sterilization with reference to the following terms.

Terms for sterilization:

1. Sterilize by autoclave at 134°C while 18 minutes for endodontic instruments and filling.
2. Do not use high pressure steam sterilizer which heats more than 200 °C including drying process.
3. When reusing the instrument, wash away foreign substances completely and sterilize.
4. Regarding use of medical cleaning agent, follow the instruction manual by its manufacturer strictly.
5. Dispose the product if damaged or decreased in performance.

10. Packagings

Sterile blister - 6 files

- Always keep this document near at hand. In case of loss, please contact dental department of the distributor for a copy.
- The specifications, structures and materials are subject to be changed without previous notices for the demands of improvement.
- The contents of this instruction manual are subject to be revised without previous notices.

11. Symbols

	Stainless steel material		Root canal preparation
	K Files		H Files
	K Reamers		Assorted package
	Sterilized using irradiation		Keep out of direct sunlight
	Quantity		Keep dry
	Consult instruction for use		Do not use if package is damaged
	Date of manufacture		Autoclavable at the specified temperature
	Use-by date		Caution
	Medical device		Batch code number
	Importer		Catalogue number
	Manufacturer		Authorize representative in the European community
			Authorize representative in the Swiss community

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